

## Advice for cleaning and disinfecting ATTENTIVE devices

## Multi-surface cleaner



- Use a microfiber rag soaked in a multi-surface disinfectant liquid without bleach.
- 2. Do not apply any liquid or disinfectant directly to the device.
- **3.** Clean gently, avoiding any product from seeping into the openings of the device.
- 4. To ensure the durability of the logo markings and preserve the color of your device, we recommend avoiding alcohol-based disinfectants





**Useful Tip!** Use a white eraser to remove scuff marks from white devices surfaces.